

T150

 - Maximum process volume capacity
190 L per cycle.

Our patented process is designed to **shred and sterilize** infectious waste by saturated steam. Shredding and sterilization are achieved in one fully **enclosed and automated**, stainless steel, system with no **intermediate waste handling**.



The contaminated waste is loaded into the upper chamber where a heavy-duty shredder reduces the waste into small pieces. Using gravity, the processed material drops into the lower treatment chamber.



After shredding, waste and all the inner parts of the machine are steam heated to 138°C (280 F) and pressurized to 3,5 bar (51 psi).



Sterilization is achieved by maintaining a temperature of 138°C (280 F) at the core of the waste for 10 minutes.



Waste are both sterilized (Microbial inactivation = 10^8 reduction of Geobacillus Stearothermophilus spores), volume-reduced by up to 80 % and weight-reduced by up to 40 % depending on the nature of the waste.



The 30 minutes duration of an average cycle process is fully automated and totally monitored. A computerized control system scans the process and automatically prints and records on a memory card a batch report at the end of each cycle with the essential sterilizing parameters for accurate record keeping.



Approved by the Higher Council of Public Health of France under the direction of the Ministry of Health and the Ministry of the Environment.

ASME CE LNE



Management System
ISO 9001:2015
ISO 14001:2015
www.tuv.com
© 2015 TUV



T150 - Maximum process volume capacity
190 L per cycle.



Technical specifications

General Characteristics




Dimensions (L x l x H)	274x200x300 Cm
Dimensions (L x W x H)	6,9x7,2x9 Ft
Total volume	484 L
Shipping weight	800 Kg
Max weight when filled with water for a special hydraulic test	1284 Kg
Stress	1 Kg/cm ²
Steam pressure	8 Bar
Max steam flow	65 Kg/h
Compressed air	6 Bar
Electricity 380 V / 3-Phase	8 kW

Operating Characteristics

Treatment temperature	130 - 177 °C
Average cycle time	30 Minutes
Max loading volume capacity	190 L
Average process volume capacity	150 L
Average waste density	100 - 150 Kg/m ³
Average process weight capacity	15-23 Kg/cycle 33-51 Lb/cycle
Sterilization: Reduction of Geobacillus Stearothermophilus spores	10 ⁸
Waste volume reduction	80%
Max waste weight reduction (depending on the nature of the waste)	40%

Consumption / cycle

Steam	9 Kg
Electricity	1,4 kWh
Water	15 L

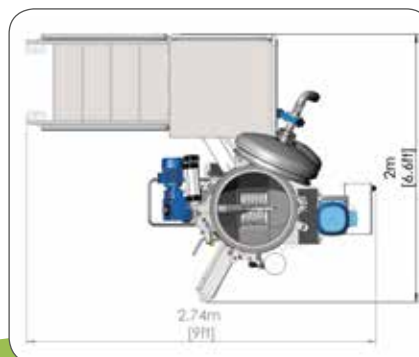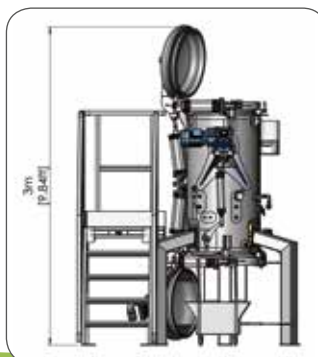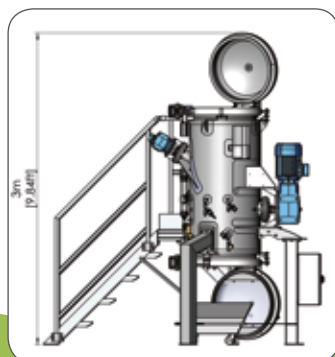

28 rue Sébastopol
59100 ROUBAIX - FRANCE

Tél. +333 20 70 98 65

contact@ecodas.com
www.ecodas.com

